

FLUSHTECHGI



Simultaneous Port Flushing



Pulsating Flush Technology



Flushing Sensor with Alarm



Manual or Auto Flush Settings



High Capacity Flushing



Reporting Capabilities



No Syringing Required



System Decontamination Cycle



Serial #	Date	Time	Event Time	Event Type
0000000	11/15/13	03:16:26PM	0075	Decontam
298567	11/15/13	03:16:26PM	0010	Dual Chan
98567	11/15/13	03:16:26PM	0010	Dual Chan
567	11/15/13	03:16:26PM	0010	Dual Ch
9	11/15/13	03:16:26PM	0010	Dual
1/15/13	03:16:26PM	0010		D
13	03:16:26PM	0010		

Automated Endoscope Flushing No Manual Syringing





The FlushTech GI™ is a powerful, easy-to-operate endoscope flushing aid that uses single and dual flushing modes to move chemistry and water through all of the channels of an endoscope. Pulsating flush technology works to help remove bio-burden from the endoscope

www.knightequip.com

before high-level disinfection in an automated endoscope reprocessor. It includes built-in programmable settings, allowing users to set flushing parameters that will meet specific conditions for achieving optimum cleaning results and high capacity processing.

KNIGHT®

Product Specifications

Specification	Description
Dimensions	7" H x 8" W x 7" D (17.8 cm x 20.3 cm x 17.8 cm)
Case Rating	IPXO
Flush Pump Flow Rate	.75 GPM (2.84 LPM)
Flush Pump Suction	Maximum suction height—6ft. (1.82m)
Power Supply/Voltage	  Wall Mount Type, IN: 110-240 VAC 1A 50-60 Hz Out: 12 VDC 1.5A
Chemical Compatibility	Flush Pump—Industry standard high level disinfectant, enzymatic and other detergents in diluted form. Do not use Isopropyl Alcohol with the flush pump.
Flush Tubes / Hook Up's Compatibility	Industry standard enzymatic and other detergents in diluted form. Do not use Isopropyl alcohol with the flush pump.
Unit Weight	9.4 lbs. (4.3 Kg)
Approvals	  CAN/CSA-C22.2 No. 61010-1-04 UL Std. No. 61010-1 (2nd Edition)

Ordering Information


Part Number	Description
7260030	Knight EndoFlateGI with power supply adapters for North America, North Asia, Europe, UK, China, Australia

Report Sample

Serial #	Date	Time	Event Time	Event Type
0000000	11/15/2013	03:16:26PM	0075	Decontamination
3298567	11/15/2013	03:16:26PM	0010	Single Channel
3298567	11/15/2013	03:16:26PM	0010	Single Channel
3298567	11/15/2013	03:16:26PM	0010	Single Channel
3875649	11/15/2013	03:16:26PM	0010	Single Channel
3875649	11/15/2013	03:16:26PM	0010	Single Channel
3875649	11/15/2013	03:16:26PM	0010	Single Channel
2783645	11/15/2013	03:16:26PM	0010	Single Channel
2783645	11/15/2013	03:16:26PM	0010	Single Channel
2783645	11/15/2013	03:16:26PM	0010	Single Channel
2836243	11/15/2013	03:16:26PM	0010	Single Channel
2836243	11/15/2013	03:16:26PM	0010	Single Channel
2836243	11/15/2013	03:16:26PM	0010	Single Channel
0000000	11/15/2013	03:21:13PM	0075	Decontamination
3298567	11/15/2013	03:21:49PM	0010	Single Channel
3298567	11/15/2013	03:21:49PM	0010	Single Channel
3298567	11/15/2013	03:21:49PM	0010	Single Channel

Accessories

The FlushTech GI comes with a complete set of endoscope irrigation flushing tube components for the most common flexible endoscopes.

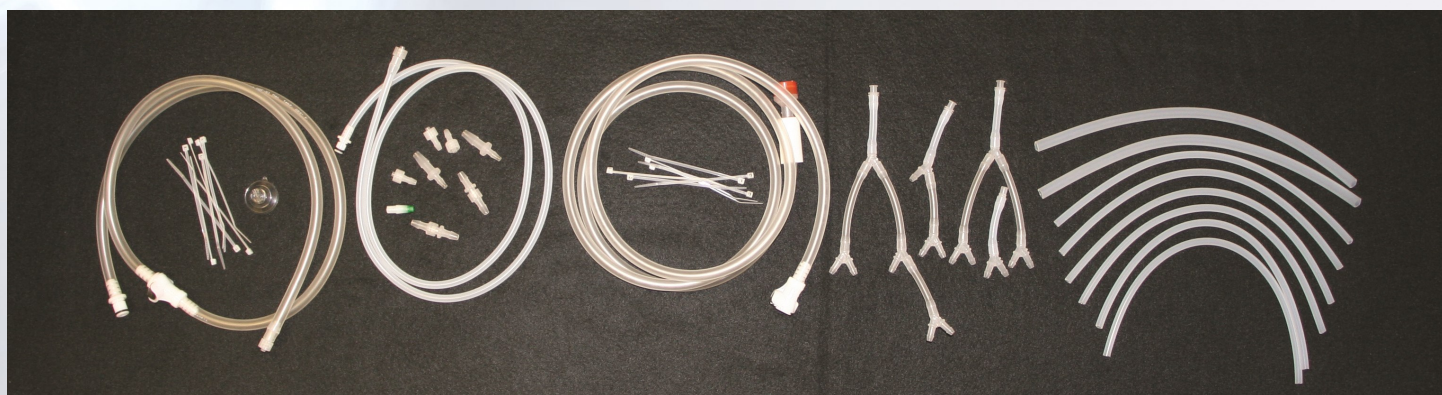

KNIGHT®

20531 Crescent Bay Drive
Lake Forest, CA 92630 USA

Toll Free: (800) 854.3764
Tel: (949) 595.4800
Fax: (949) 595.4801
www.knightequip.com

Knight Canada
Tel (905) 542.2333
Fax (905) 542.1536

Knight North Asia
Tel +82 (2) 3481.6683
Fax +82 (2) 3482.5742

Knight Europe
Tel +44 (1323) 514.855
Fax +82 (1323) 514.828

Knight South Asia
Tel +65 9170.0984
Fax +65 6489.6723